


PULSE™

Musician Filtered Earplugs

* Available in 10dB, 15dB, 25dB, and 30dB (solid)



● P10 ● P15 ● P25 ● P30
dB10 / NRR11 dB15 / NRR13 dB25 / NRR23 dB30 / NRR23

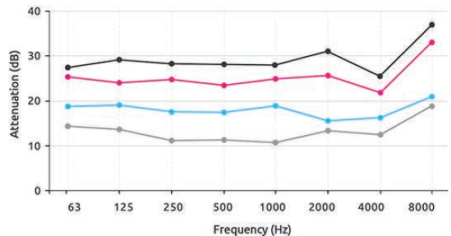

CUSTOM-FIT EARMOLDS INSERTION & REMOVAL

1. Place the canal tip of the mold into your ear canal. Wiggle the eamold to provide a good fit.
2. Then gently rotate the mold backwards towards the rear of your head, while pressing the mold into your outer ear. This moves the tip down the canal.

INSERTION



1. Pull up gently on the top-rear of your ear to break the seal.
2. Pull the top of the mold out of the ear and rotate the mold forward. This removes the tip from the ear canal.

REMOVAL



Watch the video >>>>>>>>

MANUFACTURED BY:

Microsonic, Inc. | Ambridge, PA 15003, USA
1.800.523.7672 | support@microsonic-inc.com
microsonic-inc.com

FDA REGISTRATION
NUMBER: 2523819

FDA LISTING
NUMBER: D20061



microsonic-inc.com
support@microsonic-inc.com
800-523-7672

How to Insert & Remove Custom Earplugs



Solid Earplugs



PULSE™ Earplugs

MICROSONIC®
CLEARLY THE SOUND CHOICE



Guide for Proper Use and Care
Swim Plugs, Sound Defenders,
PULSE™ & LifePlugs™
M2000/M35 Silicone

SWIM PLUGS



Microsonic NEON-LITE™

Swim Plugs protect the ear from water after infection or surgery, or in the case of chronic conditions.

Cleaning is recommended after each use. Wipe gently with a facial tissue dampened with Microsonic's SafeTouch® cleaner. Allow to dry before use.

CAUTION: Do not dive while wearing swim plugs.

SOUND DEFENDERS



Sound Defenders from Microsonic reduce workplace and environmental noise. Select from 2 materials, finishes, and various earmold designs.

Cleaning is recommended after each use. Wipe gently with a facial tissue dampened with Microsonic's SafeTouch™ product or water.

Do not immerse in water if there is a vent hole present, as it will adversely affect performance.

A drop of our EZ-Gel™ will ease insertion and create a better seal.

Passguard™ - NRR 27d
M2000™ - NRR 26dB

LIFEPLUG™



LifePlugs™

Take the edge off of everyday life.

Our new custom fit extended-wear earplugs tone down the auditory disturbances of life.

LifePlugs™ Standard - Vented & pressure free. Perfect for yoga, exercise, meditation...anything!

LifePlugs™ Sleep for a quiet, comfortable sleep.

Hypoallergenic & Antibacterial
Thousands of color combinations

LifePlugs™ *Standard* - NRR 17dB [vented]
LifePlugs™ *Sleep* - NRR 25dB [ventless]

MUSICIANS EARPLUGS



Our new PULSE™

earplugs are paired with Microsonic *P-Series* filters. These filters provide equal sound reduction across the frequency spectrum, something that is essential to musicians.

Available in a variety of colors, PULSE™ earplugs protect against damaging sounds, while allowing complete clarity and fidelity.

Choose either canal or 1/2 shell design.

Cleaning is recommended after each use. Wipe gently with a facial tissue dampened with warm water and mild soap.

Do not immerse PULSE™ earplugs in water.